

CLICK / SCAN



FOR MORE INFO

# PRG Barrier Coat

## BIOACTIVE LIGHT-CURE PROTECTIVE SHIELD

Smart solution to revolutionize the preventive protocol with two key benefits

- **Seal against Dentin Hypersensitivity** – immediate and lasting sensitivity relief
- **Protect against Dental Caries** – effective prevention and management of caries

A single application of PRG Barrier Coat provides immediate and lasting sensitivity relief. The durable coat protects teeth from external irritants and exhibits resistance to daily toothbrushing.

The patented bioactive S-PRG filler technology provides a unique combination of 6 beneficial ions to provide a fluoride rich environment with an anti-plaque effect to protect against caries overtime.



Available in:

- **PRG Barrier Coat Mini-Kit**  
Base 0.02mL x 8pcs, Active 0.9ml  
Disposable brush x 8, holder x 1

## CLINICAL APPLICATIONS

- **Dentin hypersensitivity due to**
  - Tooth wear – Attrition, Abrasion, Erosion, Abfraction
  - Gingival recession
- **Prevent and manage dental caries**
  - Early caries lesions (white spots)
  - Exposed root surfaces
  - Around orthodontic brackets
  - Interproximal area in crowded teeth
  - Teeth with clasps of removal partial dentures
  - Newly erupted permanent molars
  - Special needs and medically compromised patients

## FEATURES

- ✓ **Immediate and lasting dentin hypersensitivity relief**
  - Effective up to 6 months with a single application
  - Increase patient comfort & quality of life
- ✓ **Simplified clinical procedure**
  - Fast and easy application
  - Invisible, only 15µ thin for better aesthetics
  - Self- adhesive, HEMA-free, Acetone-free and Alcohol-free
- ✓ **Prolonged protection**
  - Durable and resistant against daily tooth brushing
- ✓ **Bioactive to protect against caries**
  - Minimize plaque formation
  - Reduce cariogenic bacteria
  - Promotes remineralization with fluoride release and recharge

## DIRECTIONS FOR USE

- Add 1 drop of Activator to the Base & mix
- Paint on clean tooth surface in a single direction & leave for 3 sec
- Light cure for 10 sec (with LED curing light)

## RELATED PRODUCTS

- **BeautiSealant**  
– Bioactive Pit & Fissure Sealant

